




# Sure Digital Cup and its proprietary Resulting App is the future of drug testing now

Our proprietary Resulting App helps remove prejudice, and integrates with our DIMS Laboratory Management System for automatic capture of history and reporting.

## What is the problem? — Result Lines are Hard to Judge


Sometimes a very faint line in the test region is a key differentiator between a positive and negative result. This is almost impossible to decipher with a human eye and could require expertise and training.

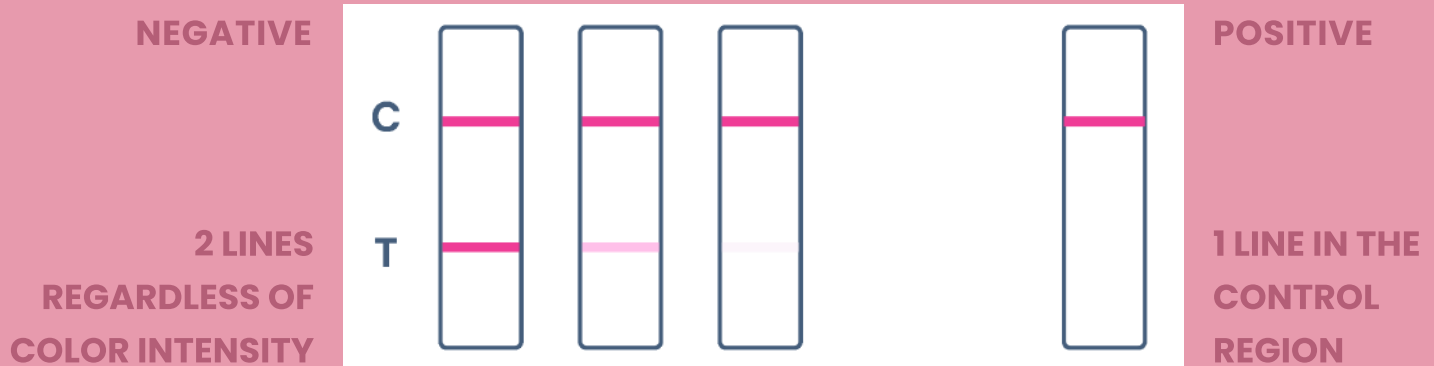

The results of ambiguous tests are left open to interpretation and prejudice of the staff members potentially carrying donor bias leading to false reporting, an issue prevalent in the industry.

Currently results are either written down on paper, manually entered into another system or both. This process adds manual entry error issues in both cases.

# Introducing the DIMS SURE App

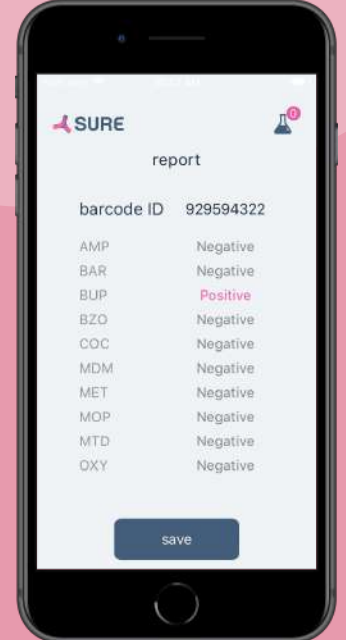

## 1. Sure Digital Cup Specimen Order Entry

- The operator enters donor information one time per client prior to taking a sample in the DIMS Product Suite.
- All scanned results flow back to DIMS for that donor, program, and organization.

## 2. Specimen Specific Barcode is Auto-Generated


- The order is automatically barcoded for quick processing of samples.
- Give the barcoded cup to the donor for a specimen.
- Digitally scanned cup provides results to the order via reading the barcode.

## 3. Sure App Scans the Donors Cup Results Optically

- The proprietary cup and scanning device work together to read sample results.
- Sure app records each sample scan for historical review.

## 4. Digital Results Create Automated History and Reporting

- All results flow back into DIMS for historical keeping.
- Donor and program results can be reported to a host of entities.

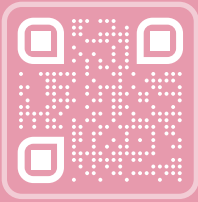

# Introducing the DIMS SURE App

Advanced Processing of the Sure Digital Cup  
via the Sure App Device.

## 1 Sure Digital Cup utilizes a proprietary 14 Panel cup.

AMP 1000ng/ml, BAR 300ng/ml, BUP 10ng/ml, BZO 300ng/ml, COC 300ng/ml, MET 1000ng/ml, MDMA 500ng/ml, MTD 300ng/mL, MOP 300ng/ml (OPI), OXY 100ng/ml, PCP 25ng/ml, THC 50ng/ml, ETG 500ng/ml, FEN 100ng/ml; Validities: SG,CRE, pH

- Sure 14 Panel Digital Cup is Forensic Use Only.
- Digital Cup includes Fentanyl and ETG.
- 14 Panel is configured specifically for the Sure App. (Alternate cups do not work)

## 2 Barcode is generated per donor sample.

- Donors sample specifically associated with testing and cup for accuracy.
- Barcode reduces handling of specimen.

## 3 Sure App scans donors results optically.

- Sure App first ID's the barcode for donor association.
- Sure App digitally reads all 14 panel results off the Sure Cup test strips.
- Digital scan eliminates common human errors related to reading and interpreting results.

## 4 Result History and Reporting is automated.

- The results are collated in a historical database.
- Historical results are available in multiple configurations centered around the client, the program, or the lab.
- Reports of that result or history of results can be downloaded, printed, or emailed.

## Contact Us

 [dims@datagainservices.com](mailto:dims@datagainservices.com)

 (606) 391 3467  <https://dimsservices.com>